

FACE MASK AFNOR SPEC S76-001 : 2020

FM/506 - 1

UNS 1 :

INDIVIDUAL MASK FOR USE BY PROFESSIONALS IN CONTACT WITH THE PUBLIC



This device is neither a medical device within the EU Regulation 2017/745 (surgical masks) nor personal protective equipment within the meaning of EU Regulation 2016/425 (FFP2 type filtering masks). This mask is strictly adapted to the health crisis linked to COVID 19. It is designed for professional use and is exclusively reserved for Non-Sanitary Uses. It is designed to prevent the projection of droplets. The use of this mask is in strict application of the measures related to quarantine, work organization measures and barrier gestures. This individual mask is designed to be offered to professionals who have to meet a large number of people during their activities (cashier hostesses, law enforcement officers). It is not intended to be used by healthcare personnel in contact with patients.

The mask can be worn for a maximum of 4 hours. The wearing test of 4 hours was carried out by us.



Guaranteed filtration
Tested 50 washing cycles

USER MANUAL

WARNING

To protect your health and the health of others, it is very important to follow these instructions.

- Wear this mask when you are in contact with people other than those you live with.
- Always make sure the mask fits snugly and covers your mouth and nose.
- Caution: If you are ill, this mask does not fit. Ask your doctor for advice.
- This mask is not intended for use by nursing staff.
- **This mask does not replace barrier gestures** (regular hand washing, physical distancing, reducing contact with other people). **It adds a physical barrier, to be used in particular when you are in close contact with other people.**

WHAT IS THE RIGHT WAY TO WEAR THE MASKS?

Before you put it on:

01. Before touching the mask, wash your hands with soap and water or a hydro-alcoholic solution.
02. Inspect the mask and make sure there are no holes, tears or damage
03. It is recommended to wear the mask on bare skin, avoiding contact with hair.
04. Do not alter the mask in any way.

To put it on:

05. Hold the mask by the elastic straps.
06. Place it at the level of your mouth.
07. Put the elastic straps behind your ears.
08. Adjust the mask so that it covers your nose, mouth and chin.

When you wear it:

09. Avoid touching and moving it.
10. Never put it in a waiting position on your forehead or chin.

When do we need to change the mask?

11. When you wore the mask for four hours.
12. When you want to eat or drink.
13. When it becomes hard to breathe.
14. When the mask is humidified.
15. If the mask is damaged.
16. If the Mask is deformed and no longer fits properly against your face.

To remove it:

17. Wash your hands with soap and water or a hydro-alcoholic solution.
18. Unhook the elastic straps to remove the mask from your face.
19. While waiting to wash it, isolate it in a plastic bag.
20. If the mask is deformed or worn out, throw it away immediately in a closable garbage can.
21. Finally: Wash your hands again with soap and water or hydro-alcoholic solution.



HOW TO MAINTAIN THE MASK?

Before opening:

22. Store the mask in its original protective packaging in a dry place away from direct sunlight. Store it in a maximum humidity level of 75% and at a temperature between -2°C and +55°C in order to preserve all its characteristics.

For washing:

23. Machine wash the mask with detergent. Choose a washing program that includes at least 30 minutes at a minimum temperature of 60°C (140°F). You can put it in the machine with your washing. Wash separately to avoid risk of color bleeding.
24. After touching the mask, wash your hands with soap and water or hydro-alcoholic solution.
25. This mask has been tested for 50 washes. After 50 washes, the filtration of the mask is no longer guaranteed.
26. When the machine is finished, take the mask out and

re-fold the folds without using force.

After washing:

27. Dry the entire mask (the inner layers must also be completely dry).
28. To do this, you can: use a tumble dryer (making sure to clean the dryer's windows and wash your hands afterwards), use a hair dryer, or dry your mask in the open air on a clean and disinfected surface.
29. Steam iron your mask at the temperature indicated by the manufacturer.
30. As soon as the mask is dry, store it in a clean, airtight package (such as a plastic or freezer bag).

SAFETY

31. This mask has demonstrated a filtration capacity greater than 90% for 3-micron particles emitted by the person wearing the mask.
32. The breathability of the mask has been measured at > 96 L m⁻².s⁻¹ and must allow it to be worn for a period of 4 hours.

WARNING

Despite all the care brought in the manufacture process of these masks, it is the sole responsibility of the user, and under no circumstances the manufacturer or supplier, to ensure that the mask is suitable for the characteristics of the environment in which it is to be used. The recommendations and conditions of use described above must be scrupulously followed.

GOOD TO KNOW :

During the first wash, the mask can reduce by 5 to 10%. Even after this, the MedicalZ face mask covers the average morphology of the population and still provides adequate protection.

COMPOSITION: Outer fabric: 100% Washed Cotton - Inner loop mesh: 100% Polyester - Elastics: 20% Polyamide 20% Elastane - 60% Polypropylene.

Materials whose performance has been measured by the laboratories of DGA Maitrise NRBC, 5 rue Lavoisier, 91710 Vert-le-Petit (report RP/20-3222/DGA MNRBC/2000305/NP) and IFTH, 93 Chemin des Mouilles, 69134 Ecully and which withstands 50 washes (report N°2020-04-21115-2/IFTH/08-06-2020)

ZI N°1 - 14 rue Georges Cuvier
37550 SAINT-AVERTIN
Tél. : +33 (0)2 47 71 33 33
Fax. : +33 (0)2 47 71 33 34

Email : masqueunsfrance@medicalz.com

www.medicalz.com